**SOLUCIÓN DE PERÓXIDO DE HIDRÓGENO**  
**HYDROGEN PEROXIDE SOLUTION**  
**SOLUTION DE PÉROXYDE D´HYDROGÈNE**

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**ENGLISH:** This leaflet contains all the instructions necessary for using the peroxide system correctly as well as important precautionary measures. Please read this sheet carefully and save it for future reference. Incorrect maintenance of the contact lenses can cause serious eye problems.

IRIS LINE is a single phase hydrogen peroxide cleaning system, that provides the optimum cleaning, disinfection, hydratation and protein removing of all type of contact lenses (except color lenses). The solution is free of preservatives and should always be use with its special lens case with platinum catalyst disc to neutralize the peroxide.

**INSTRUCTIONS FOR USE:** Before handling your lens, carefully wash your hands and dry them with a clean towel. Only use the lens case included in the IRIS LINE Peroxide System box. Fill the lens case with IRIS LINE Peroxide solution up to the dosage line. Place each lens in the right compartment, being careful not to crush them when closing the compartment, and immerse it into the solution. Carefully close the lens case and keep it in a vertical position. Don´t shake the lens case. When the product functions correctly, oxygen bubbles are generated once the platinum star, in the bottom of the lens case, comes into contact with the solution. If this is not the case, don´t put the lenses into the solution. Leave the lenses immersed in the solution for at least 6 hours or overnight. Before inserting the lenses, rinse them with IRIS LINE Eye Wash or a physiological solution (never with tap water). Once the disinfection process is complete, rinse the lens case and leave it to air dry. The lens case must be changed once a month or whenever a new bottle is opened, in which case a new neutralizing lens case is supplied. Do not remove the lid of the lens case whilst disinfecting the lenses (when there are oxygen bubbles). Do not use the lens case with platinum neutralizing star with any other solution other than IRIS LINE Peroxide System.

**PRECAUTIONS:** Once the bottle is open, throw away the remaining solution after 60 days. After each use, always throw away the remaining solution in the lens case. Close well the bottle after each use, and keep it away from heat sources, at a temperature between 5°C to 30°C. Do not touch the dropper to avoid contamination. Do not use the product if the seal of the bottle is not intact or open. This product is sterile until the bottle is opened. If lenses are left in the same solution for more than 24 hours, repeat the cleaning process. Do not use after the expiration date indicated on the packaging and label. Make sure that the platinum star is well fixed to the bottom of the lens case.

**WARNINGS:**  
**IRRITANT PRODUCT, DO NOT INGEST, DO NOT PUT IN DIRECT CONTACT WITH THE EYES OR MUCOSAE.**  
 The peroxide solution is **IRRITANT** before being neutralized in the lens case with the platinum catalyst star.  
**NEVER RINSE DIRECTLY THE LENSES WITH THE SOLUTION IRIS LINE PEROXIDE SYSTEM AND AVOID TO PUT IT IN DIRECT CONTACT WITH THE EYES. DO NOT INGEST THE SOLUTION.**  
 Never insert lenses directly into the eyes after wetting them with IRIS LINE Peroxide System, if they have not first undergone the neutralization process with the platinum catalyst star lens case.  
 Never use the IRIS LINE Peroxide System solution, with another lens case different from the one contained in the packaging.  
 Do not use if you are allergic to any of the ingredients. In case of irritations or other problems, remove your lenses and contact your specialist.  
 Keep out of reach of children.

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